REMSTIM®
RAPID EYE MOVEMENT STIMULATOR
3000

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Congratulations!

**With your new REMSTIM 3000
you have the ideal partner for more
emotional balance and greater mental performance found.**

Operating instructions for the

REMSTIM 3000

Status 06 June 2023: Subject to change without notice

Content

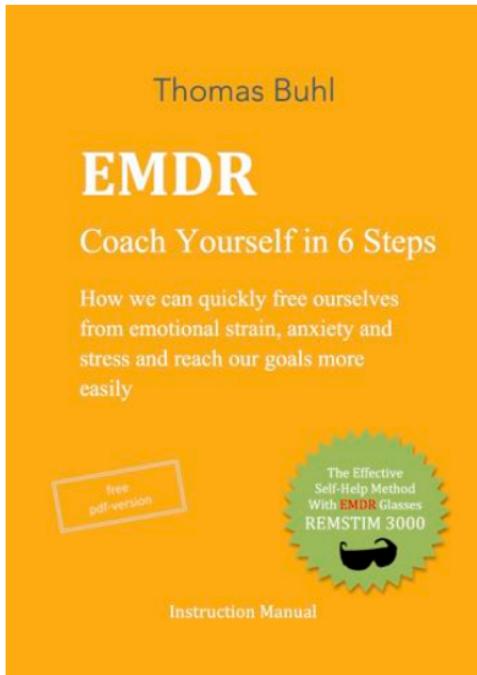
Before first use	24
Important information	25
Operation.....	27
Application.....	29
Use the REMSTIM 3000 in 6 steps.....	30
Keeping the REMSTIM 3000 in top shape	33
Warranty.....	33
Note according to battery law (BattG)	33
Replacing the battery	34
Electromagnetic fields	38
Notes on the disposal of old appliances and electronic scrap	38

Customer service

Thomas Buhl Coaching Instruments
At the badger building 59
13503 Berlin
Germany

Service hotline: 030-39 82 14 19
E-mail: info@remstim.com
Internet: www.remstim.com
Device type: REMSTIM 3000

Before first use

The REMSTIM 3000 - Rapid Eye Movement Stimulator 3000 is specially designed to stimulate the user to move his eyes back and forth horizontally in a steady manner. This type of eye movement is used, among other things, in rapid eye movement stimulation (also called bilateral hemispheric stimulation). The stimulation of eye movement by the REMSTIM 3000 can be adjusted by the user in terms of deflection and speed of the running light points.

Please read the operating instructions carefully and completely before using the REMSTIM 3000 for the first time and keep them in a safe place. If the REMSTIM 3000 is passed on to a third party, this manual must also be handed over.

In this instruction manual you will find the short instruction "Using the REMSTIM 3000 in 6 steps". In addition, it is recommended that you download the comprehensive instructions "EMDR self-coaching in 6 steps" free of charge online at www.remstim.com.

In it, you will find detailed instructions on how to make the most of EMDR self-coaching.

Important information

1. The REMSTIM 3000 is an eye movement stimulation device. The use of the REMSTIM 3000 for the stimulation of eye movements is understood as a performance-enhancing mental training. It is not a diagnostic procedure, medical device, medical therapy or any other kind of healing, does not treat diseases or symptoms of diseases. No healing promises are made, so that false hopes are raised in the user.
2. The use of REMSTIM 3000 is not intended to replace the work of a psychotherapist, physician or alternative practitioner. Therefore, an ongoing treatment should not be interrupted or discontinued, or a future necessary one should not be postponed or omitted altogether.
3. The use of the REMSTIM 3000 in no way cancels medical orders. The responsibility lies entirely with the user.
4. The description of physical, emotional and mental states - in particular through the use of words such as stress, strain, worry, sorrow, restlessness, anxiety, tension and similar terms - is exclusively a description of the sensitivities of healthy people within a natural range of variation. Under no circumstances is the impression to be created that the depicted sensitivities go beyond a certain degree of impairment and attempt to describe or even represent illnesses, suffering, physical damage or pathological complaints.
5. A prerequisite for the user is - as is usually the case with coaching - a normal mental and physical resilience.
6. Use of the REMSTIM 3000 is not suitable at any time for persons who suffer or have suffered from the following diseases in particular: Epilepsy, eye diseases, brain organic diseases, addictive diseases, dissociative disorders, ego disorders.
7. It is the free responsibility and decision of the user to start, continue or stop using the REMSTIM 3000.
8. The use of the REMSTIM 3000 may cause temporary intense emotions.
9. Only use the REMSTIM 3000 for the purpose for which it was made. Note that when mounted, the

- REMSTIM 3000 completely fills your field of view and completely blocks the view of your surroundings.
- 10.The REMSTIM 3000 is intended for personal self-application while seated or lying down. Up to 3 consecutive applications of 30 seconds each are recommended.
 - 11.If the eye movements are perceived as unpleasant, especially during the first applications, it is necessary to optimize (lower level) the personal device settings of deflection and/or speed. In the rare case of dizziness, nausea or eye pain, discontinue REMSTIM 3000 and adjust settings.
 - 12.Focusing of the (running) light points is hardly possible and not intended, nor is it a necessary condition for successful application.
 - 13.The REMSTIM 3000 is not suitable for use by children and must be kept out of the reach of children.
 - 14.The materials used in the manufacture of the REMSTIM 3000 may cause an allergic skin reaction in susceptible users. In this case, consult a physician immediately.
 - 15.If the manufacturer's instructions are observed when using and handling the tightly sealed lithium button cells, they are harmless or non-hazardous. If there is no hypersensitivity to certain substances (allergic reactions), skin contact with the lithium button cells is harmless.
 - 16.Do not expose the REMSTIM 3000 to extreme weather conditions (rain, snow, sunlight, wind).
 - 17.Protect the REMSTIM 3000 from water.

Operation

The REMSTIM 3000 always starts with the last settings used. Three downward running yellow light dots signal the start of the 30-second application. Three yellow light points running upwards signal the end of an application.

Switch on:

The REMSTIM 3000 is started by pressing the left key. It stops automatically after 30 seconds and can then be restarted by pressing the left key again.

Note: The REMSTIM 3000 cannot be stopped during the 30-second application. To interrupt the stimulation during the current application, put down the device or close your eyes.

Adjusting the deflection range of the eye movement: A small black icon consisting of two vertical arrows pointing in opposite directions, representing adjustment or range.

The eye deflection level defines how far the eyes are alternately guided to the left and right side during an application. The REMSTIM 3000 offers 4 deflection levels to choose from during application for optimal comfort.

The deflection range of the eye movement is set by pressing the left key during operation. The 4 possible deflection levels can be switched through by repeatedly pressing the left key. Two green light dots mark the selected deflection range by flashing rapidly several times. Afterwards, a 30-second application automatically starts again:

Deflection stage	Deflection range
1	+/- 8 LEDs
2	+/- 10 LEDs
3	+/- 12 LEDs
4	+/- 15 LEDs
1 etc.	+/- 8 LEDs etc.

Tip: For optimum effects, select the highest deflection level that you find comfortable during an application. Increasing to level 4 is recommended, assuming tolerance.

Note: Focusing of the (running) light points is hardly possible and not intended, nor is it a required condition for successful application.

Setting the speed:

The speed of the stimulation can be adjusted at any time during an application. This is done by regulating the tempo of the moving light signal.

The speed is set by pressing the right key during operation. The three possible speed levels are switched through by repeatedly pressing the right key:

Speed level	Speed
1	slow
2	fast
3	very fast
1 etc.	slow etc.

Tip: For optimum effects, select the highest speed level that you find comfortable during an application. It is advisable to increase the speed to level 3, provided that it is tolerated.

At the same speed level, the perceived speed increases when a smaller deflection level is selected.

Application

1. Sit or lie down. Stop any other activity.
2. Put on the REMSTIM 3000 like a pair of standard eyeglasses.
3. Start the REMSTIM 3000 as described in the section Operation by pressing the left key and follow the light signal with your eyes.
4. After 30 seconds, the application ends automatically.
5. Continue breathing calmly and regularly during an application.
6. You can set the eye movement **deflection** and **speed** functions according to your personal condition as described in the Operation section.

Apply the REMSTIM 3000 in 6 steps.

On the following pages you will find a quick guide to help you get the most out of the REMSTIM 3000. The application of the REMSTIM 3000 in 6 steps aims to support you in coaching certain matters and issues that are personally stressing you emotionally and mentally in a particularly careful way.

Please note that for some issues and matters that involve you in a particular way, it may be advisable to consult a coach or therapist you trust in this regard. Please also read the Important Information.

In the context of the 6-step instructions, it has proven helpful to write them in the 2nd person singular. So you will be addressed as "you" on the next pages. We ask for your understanding.

Step 1: Your topic

1. What moves you the most at this moment?
2. What thoughts are currently bothering you? What is the most unpleasant image, the most unpleasant sentence?
3. What's on your mind?
4. Concentrate on the idea, the situation, the thing, the picture, the sentence, etc., what stresses you the most!

Step 2: Your emotion

1. Do you feel anything while focusing on your subject?
2. What emotion is it that you feel?
3. For example, is it: Anger, annoyance, indignation, being annoyed, fear, anxiety, sadness, loss, helplessness, powerlessness, being at the mercy of others, surprise, confusion, "being in the wrong movie", guilt, responsibility, shame, mortification, insult, degradation, disgust, hatred, disgust, disgust, "creepiness", tiredness, thirst, hunger, not feeling? Or any other emotion?

The listing represents a number of unpleasant emotions and is not exhaustive.

Step 3: Body-Scan I - Your physical sensations

1. Do you feel something in your body? Does a body echo of your emotion exist?
2. Do you feel a stirring, a physical sensation somewhere in a part of your body?
3. Do you perceive a pressure, a pulling, warmth, cold or similar in yourself?
4. Or do you perceive nothing, and it resembles a non-perception?

Step 4: The intervention (Processing)

1. Inwardly perceive your subject and your emotion(s). Feel your physical sensation (also non-sensation) now!
2. Now start an application of the REMSTIM 3000
3. Breathe calmly and regularly!
4. When the REMSTIM 3000 application is finished, take a deep breath and release as you exhale.

5. What has changed internally? Observe your emotion(s), your body feeling, when you now think of your topic from the beginning (associations?).

→ Go back to the first point of the intervention (4th step) and start the process again. Go through the process as many times as you want. For example, until you feel distanced and liberated from your issue.

Step 5: Body Scan II

1. Think about your subject. Search and feel for small residual sensations. If you find them, intervene again with the REMSTIM 3000 as described in Step 4 until you achieve the best possible result.

6. in conclusion - the anchoring

2. Now think about your topic and notice the first positive thought that comes to mind.
3. Repeat this *pleasant thought* in your mind again and again. Intervene now with the REMSTIM 3000 - choose a slow speed of the running light.
4. Breathe calmly and regularly, repeating your pleasant thought.
5. After REMSTIM 3000 application, inhale deeply and release on the exhale.

Keeping the REMSTIM 3000 in top shape

Clean the surface of the REMSTIM 3000 regularly. Only use a dry cloth (eyeglass cleaning cloth) for this purpose.

Damage to the device: If the device is defective or damaged, do not attempt to repair the device yourself. In case of damage, contact customer service (see section Customer service).

Warranty

There is a warranty within the scope of the statutory provisions.

Note according to battery law (BattG)

The REMSTIM 3000 is operated with a battery, which is included in the delivery. Therefore, according to the Battery Act (BattG), we are obliged to inform you of the following:

Batteries and rechargeable batteries must not be disposed of in household waste, but you are legally obliged to return used batteries and rechargeable batteries!

Used batteries may contain harmful substances that can damage the environment or your health if not stored or disposed of properly. However, batteries also contain important raw materials such as iron, zinc, manganese or nickel and are recycled.

You can replace the batteries after use either

- send it back to us (Thomas Buhl Coaching Instrumente, Am Dachsbau 59, 13503 Berlin)
- or in the immediate vicinity (e.g. in shops or municipal collection points) free of charge.

The distribution at points of sale is limited to the quantities customary for end-users for disposal and to those spent batteries that the distributor carries or has carried as new batteries in its product range.

The crossed-out wheeled garbage can means that you must not dispose of batteries and accumulators in household waste. The signs under the dustbin on batteries and accumulators stand for:

- Pb: Battery contains lead
- Cd: Battery contains cadmium
- Hg: Battery contains mercury

Replacing the battery

Insufficient battery power causes functional restrictions in operation and program sequence, which are switched off again when a new battery is inserted. If the performance of the supplied battery (3.0 V button cell DL1220 / CR1220 / CR 1216. Note: CR 1225 does not fit because it is too thick) decreases, replace the old battery with a new battery.

Important: Keep button batteries away from children who can easily swallow them.

Place the glasses upside down on a smooth surface. Hold the goggles by the housing.

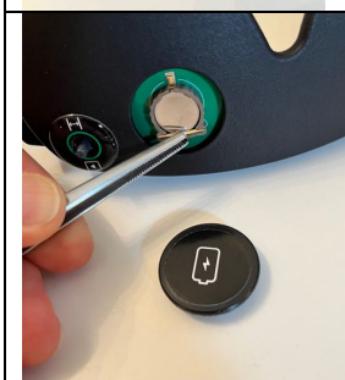
To loosen the plug that closes the access to the battery, position the fingernails of two fingers in the gap between the plug and the glasses as in the picture.

Two fingers are recommended because the stopper is stuck; they prevent a fingernail from breaking off. Now press lightly on the edge of the stopper while pulling it out with the finder nails with a light pull.

Once you remove the plug, you will see the button cell type CR 1220 3V (3.0 volt button cell DL1220 / CR1220 / CR 1216. Note: CR 1225 will not fit because it is too thick).

The button cell is held by a transverse metal strip. The battery then pops out when you bend the metal strip in the middle slightly to the edge. A pair of tweezers is helpful, with thumb and forefinger it also works.

To insert the new button cell type CR 1220 3V, hold the glasses horizontally as shown in the picture and position the button cell loosely on the button cell holder; plus contact facing up.

Press the button cell carefully and with extremely little force into the button cell holder until it clicks into place and cannot fall out. Please note that too much pressure could damage the glasses.

Finally, place the glasses upside down again on a smooth surface. Hold the glasses by the housing and position the plug over the button cell in the recess. Then, gently and with only slowly increasing pressure, push the stopper into the opening as shown in the first picture.

Electromagnetic fields

The REMSTIM 3000 EMDR device complies with all relevant standards and regulations regarding exposure to electromagnetic fields.

Notes on the disposal of old devices and electronic scrap

of these collection points.

As of March 2006, according to the Electrical and Electronic Equipment Act (ElektroG), old appliances may no longer be disposed of in household waste. This applies not only to typical electrical appliances such as televisions, washing machines and refrigerators, but also to video games, loudspeakers, computers, notebooks, MP3 players, peripherals, printers, telephones, electrically powered children's toys and other old electrical appliances. These devices can be recognized by the symbol of a "crossed-out trash can". You can find the symbol on the devices or the packaging. You can hand in electronic waste free of charge at municipal collection points, for example. Your municipal waste advisory service or the consumer centers can provide you with information about the exact location

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